




INSTRUCTIONS FOR AUDIOLOGIST

Instructions to the Audiologist for impressions: Impressions should be semi-deep to include the second bend of the ear (A) & the entire concha, helix & tragal notch (please pay particular attention to filling the upper helix/crus portion (B) of the impression). Because Surfplugs® are tight-fitting we do recommend that the material is also pushed gently into the ear to encourage a good snug fit. Please give the impressions back to the client in a box suitable for postage back to us here at Surfplugs®.

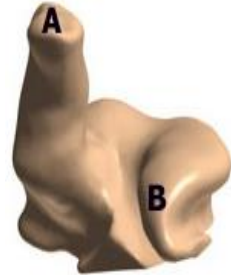

CUSTOMS DECLARATIONS:

Contents description: *Ear moulds of NO commercial value* **Value:** 1p

Surfplugs Ltd, Old School House, Elston Lane, Coplestone, Devon, EX17 5NS, UK

Registered address: Vantage Point, Woodwater Park, Pynes Hill, Exeter EX2 5FD

If you have any queries phone us on 07969 188529 or email us at info@surfplugs.co.uk